

# Second Opinion Enterprise Tutorial Video Capture

## Enterprise Video Capture

Whether you are using a desktop, laptop, tablet computer or even a smart device, there are times when it would be nice to be able to connect to a camera or capture device to capture a still image or video clip right from the browser. Enterprise Video Capture does just that, without the need to install any additional capture software on the local device. It supports built-in cameras, external USB cameras, basically any device attached to the computer that has standard Windows drivers.


### Launching Video Capture

Once you have successfully logged into the Enterprise Portal, create or navigate to the desired Patient folder. Where ever you are when the Enterprise Video Capture utility is launched, that is where the captured stills or video clips will end up.



# Video Capture Tool

When the Video Capture Tool is launched, a new Tab will be opened in your Browser. The Header Bar of the Capture Tool displays the name of the Patient the captured content will be saved in, has a Capture Still Image, Capture Video Clip and lastly a Settings buttons.

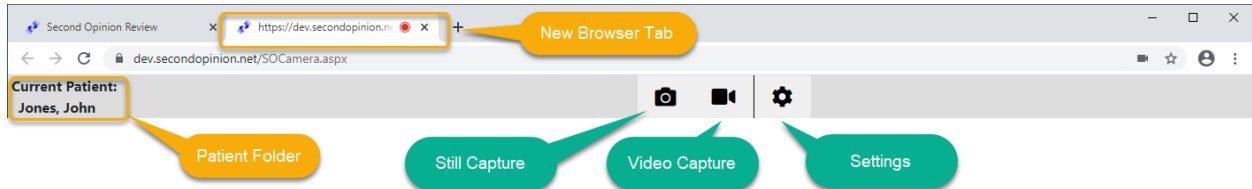

## Getting Started - Settings

Upon first use of the Video Capture Tool it is important to set the default values of the Capture Tool.

**Video Source:** Select the Video Device to Use

**Audio Source:** Select Audio Source to Use

**Note:** If you do not want to capture audio when recording a video clip either Mute the audio source or Select *No Audio Source* as the default audio source.


### Options:

**AUTO UPLOAD AFTER CAPTURE:** Automatically Save and Upload images and videos after capture

**ADD CAPTURE SHORTCUTS TO RIGHT SIDE OF SCREEN:** Adds convenient buttons to initiate capture when using hand-held devices

**USE 'SPACEBAR' TO CAPTURE STILL:** Enables Still Image capture Hot-Key, required for use with foot pedal

**USE 'C' TO START/STOP VIDEO CAPTURE:** Enables Video Clip capture Hot-Key, required for use with foot pedal

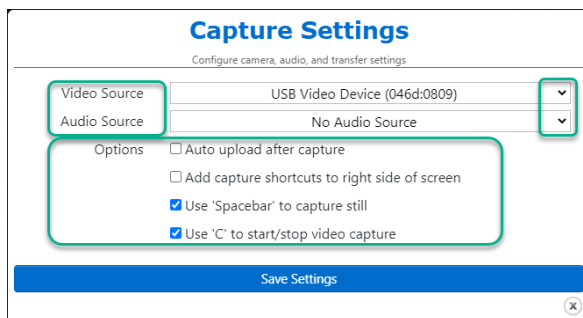

## Choose Video Source

### Capture Settings

Configure camera, audio, and transfer settings

**Video Source** USB Video Device (046d:0809)

Audio Source

Options

- USB Video Device (046d:0809)
- Integrated Camera (13d3:56b2)
- Basler GenICam Source
- No Video Source

Use 'C' to start/stop video capture

Save Settings

## Choose Audio Source

### Capture Settings

Configure camera, audio, and transfer settings

Video Source

**Audio Source** No Audio Source

Options

- Default - Microphone (Jabra SPEAK 510 USB) (0b0e:0420)
- Communications - Microphone (Jabra SPEAK 510 USB) (0b0e:0420)
- Microphone (Jabra SPEAK 510 USB) (0b0e:0420)
- Microphone Array (Intel® Smart Sound Technology (Intel® SST))
- Microphone (USB Audio Device) (046d:0809)
- No Audio Source

# Main Video Preview

The screenshot shows a video preview window for a patient named John Jones. A 'Capture Settings' dialog box is open, allowing configuration of camera, audio, and transfer settings. The settings include: Video Source (USB Video Device (046d:0809)), Audio Source (No Audio Source), and Options (Auto upload after capture, Add capture shortcuts to right side of screen, Use 'Spacebar' to capture still, Use 'C' to start/stop video capture). A 'Save Settings' button is at the bottom of the dialog. A 'Save Image or Video' button is visible in the top left corner of the video preview area. A 'Preview of Captured Images and Video Clips' is shown in a vertical strip on the left. A 'Side Capture Buttons Enabled' callout points to the right side of the interface.

# Saved Images and Video Clips

The screenshot shows the 'SECOND OPINION ENTERPRISE' interface. The user is logged in as Katie Wilson. The interface displays a list of saved images and video clips for a patient named John Jones (442313) on Wednesday, September 2, 2020. The list includes three items: '09/02/2020 17:34 image', '09/02/2020 17:34 image', and '09/02/2020 17:35 image'. A 'Saved Images and Video Clips' callout points to the list.

Page Left Intentionally Blank

For More Info or Technical Support Give Us a Call

Second Opinion Telemedicine Solutions, Inc.

20695 S. Western Ave., Suite 200

Torrance, CA 90501

310-802-6300

[www.sotelemed.com](http://www.sotelemed.com)

[corp@sotelemed.com](mailto:corp@sotelemed.com)



Tutorial Enterprise Video Capture (v1.0)

© (2020) Second Opinion Telemedicine Solutions, Inc.

Page 5 of 5