


Second Opinion Professional

How to print images with Narratives

The Second Opinion WordReport program is used to print images with narratives. WordReport uses specialized Microsoft Word templates created with the Second Opinion Report Template Designer. WordReport templates can be accessed two different ways: in the Reports section of the New Document dialog box or by running the WordReport Manager.


Option #1 – New Document Dialog Box

Select the desired images, Click the New Document button, then expand the Reports section and choose the desired report, for example, Images 4 Up.

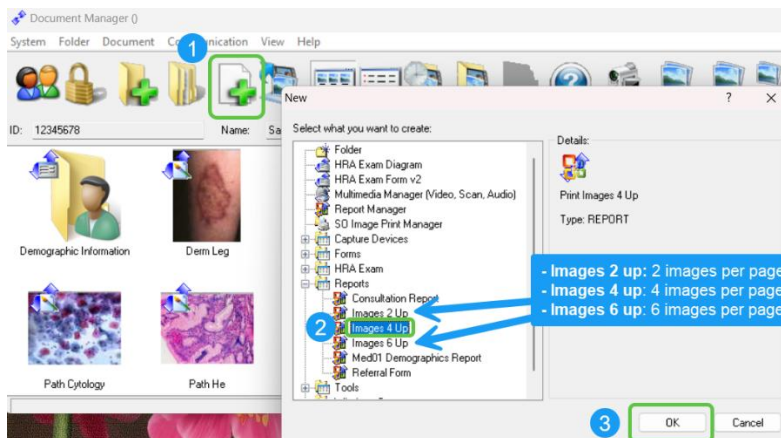



Option #2 – WordReport Manager

Select the desired images, and Click the WordReport icon on the main tool bar and choose the desired report.

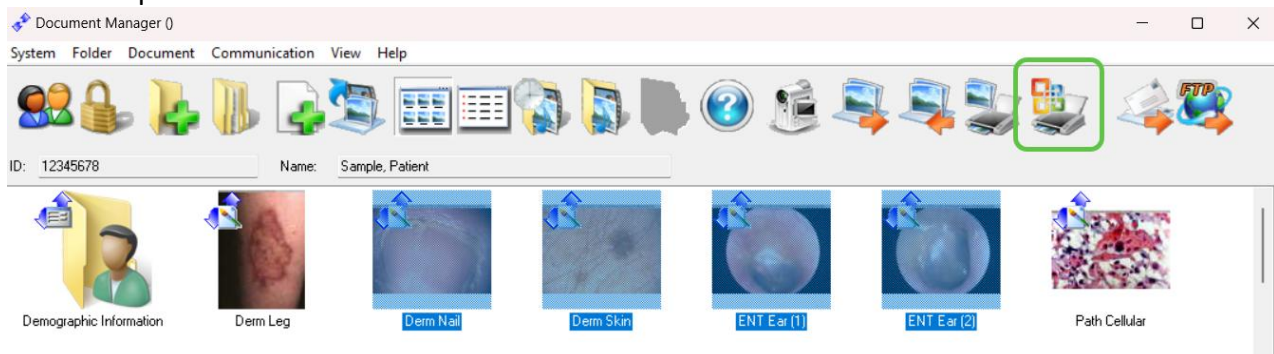


Second Opinion

Telemedicine Solutions, Inc.

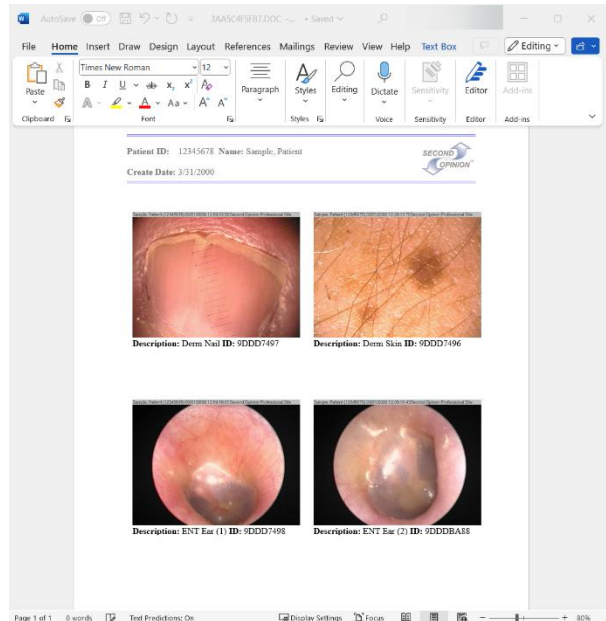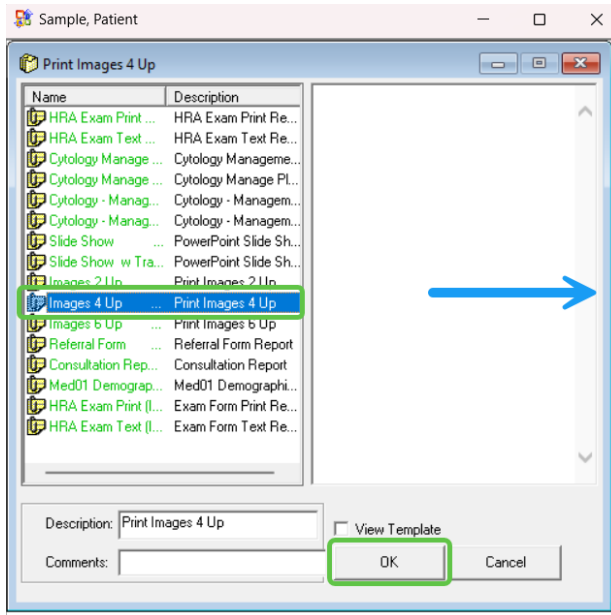


When the desired WordReport is run, the selected images and any data included in the template will be merged into a Word document. Once the merge is complete, the document can be edited and modified using all the Word tools available.

The completed document can be printed from Word by **pressing and holding CTRL + P** on the keyboard at the same time.

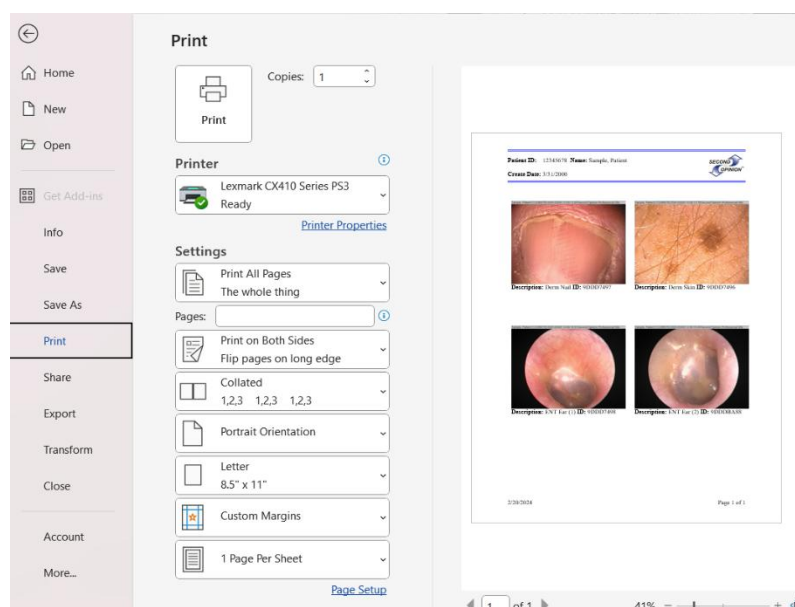



The completed document can then be saved into the Patient's Folder. This option will appear when you close the document.

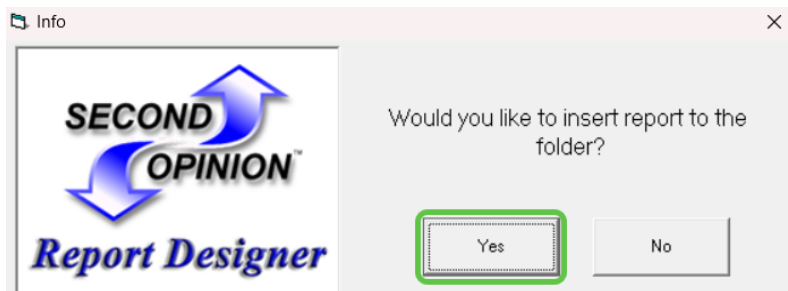

For More Information or Technical Support, Give Us a Call

Second Opinion Telemedicine Solutions, Inc.

310-802-6300

www.sotelemed.com

corp@sotelemed.com

