


Second Opinion Professional

Image Export Manager


The Image Export Manager is a simple to use tool to manage the process of exporting all or selected images from a Patient's folder to an unsecured folder outside of **Second Opinion™** Professional. During the export process, the images are converted to an industry-standard .JPEG file format. These unsecured JPEG images can be used in a variety of different ways. They can be burned to a CD/DVD or copied to a memory card/flash drive for review by persons who do not have the **Second Opinion™** Image Viewer application. They can also be uploaded to an online image library/storage system.

Note: If you are only going to export selected images, select the images before running the **Image Export Manager**.


To start the application, **Click** on the **Export Manager** toolbar button or **Click** the **New Document** button, **Expand** the **Tools** section, and **Click** on **Export Manager**.

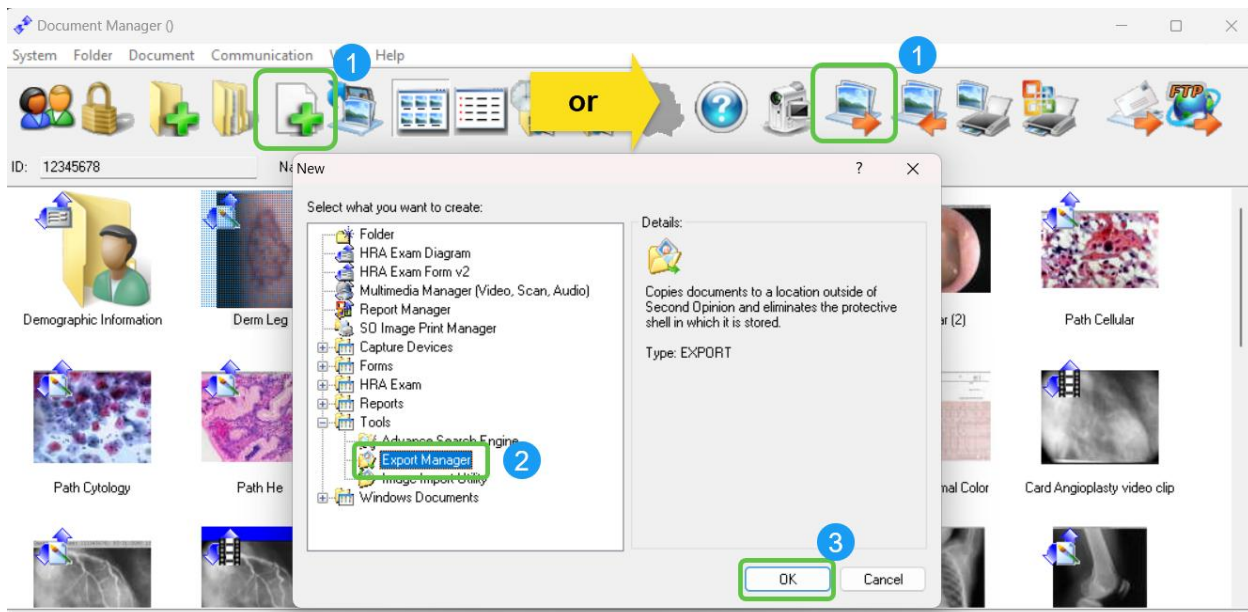

Before you can run the **Image Export Manager** you must acknowledge its disclaimer. Please review the disclaimer and if you do not understand any part of it, please call the Second Opinion Telemedicine office for clarification.

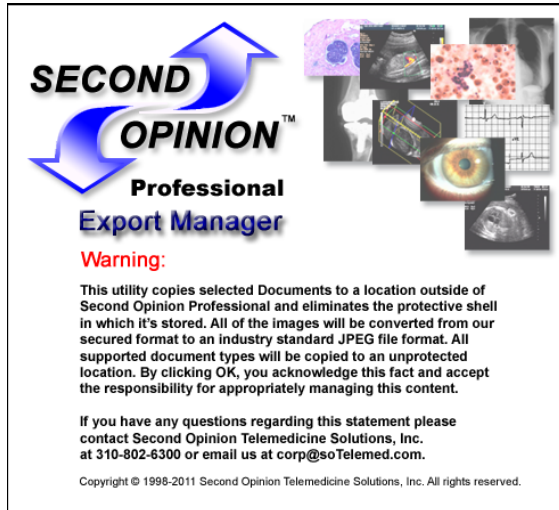

The Export Manager has the ability to export All or Selected Documents. The **Image and Stamp Options** allow you to change the image stamp on an image during the export process. When the Peer Review option is selected, the original image stamp will be removed/grayed out. The **Directory Options** section allow for the creation of sub-folders based on the Date of the content or by Sessions that have been defined. Annotations associated with an Image are merged into the image when the Annotations option is selected. Then, choose the file format of the Image and lastly choose the location where the files are to be saved.

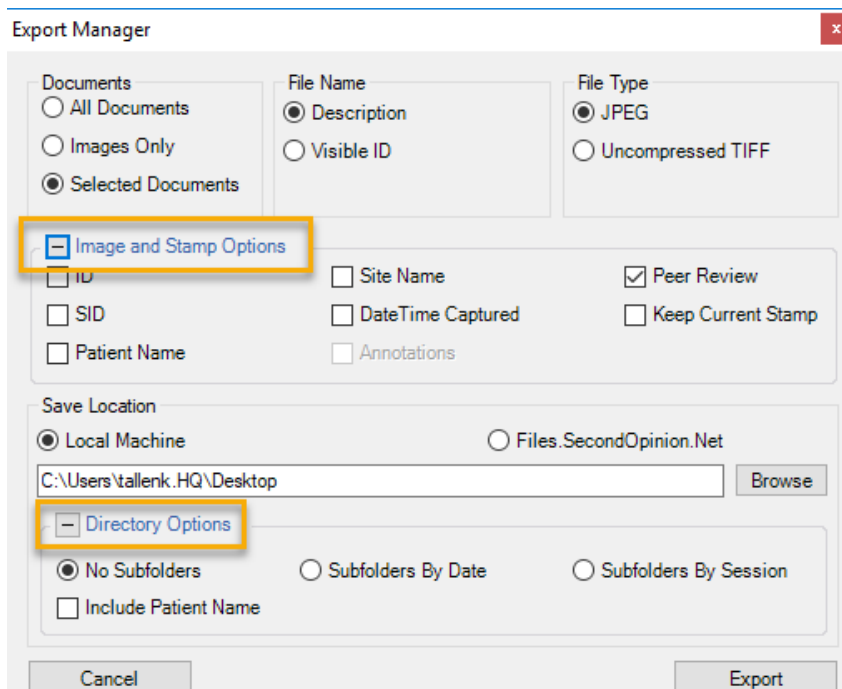



Advanced Search Integration

This Image Export Manager also works with the **Second Opinion™** Advanced Search Tool. The Advanced Search Tool now has an Export Manager Option. The query results (group of Patients) generated by the Advanced Search Tool can now be sent to the Image Export Manager and the result set can be exported in a tree-like manner according to the defined choices. For more information about the Advanced Search Integration, please refer to the Advanced Search Utility document.

For More Information or Technical Support, Give Us a Call

Second Opinion Telemedicine Solutions, Inc.

310-802-6300

www.sotelemed.com

corp@sotelemed.com

