



## Defining and using Secondary IDs

All of the Folders created using **Second Opinion**™ Professional contain two key fields, the **Visible ID** and the **Description** of the Folder. In most cases these are the **ID** and **Name** of a Patient.

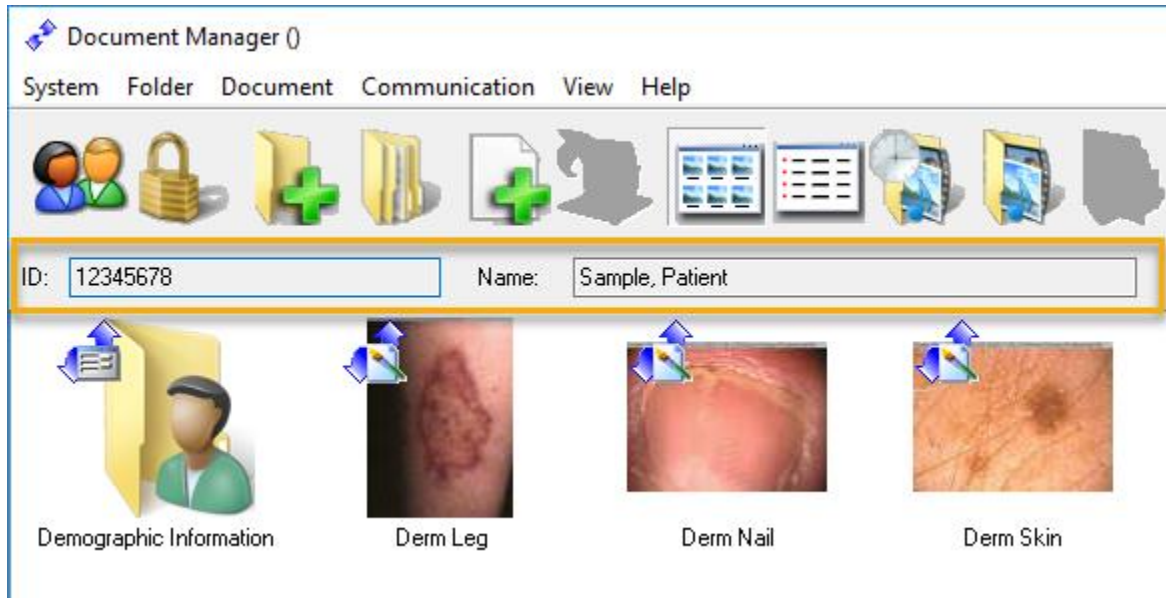


Other information related to the Patient's folder like demographic information is stored in separated form / databases.

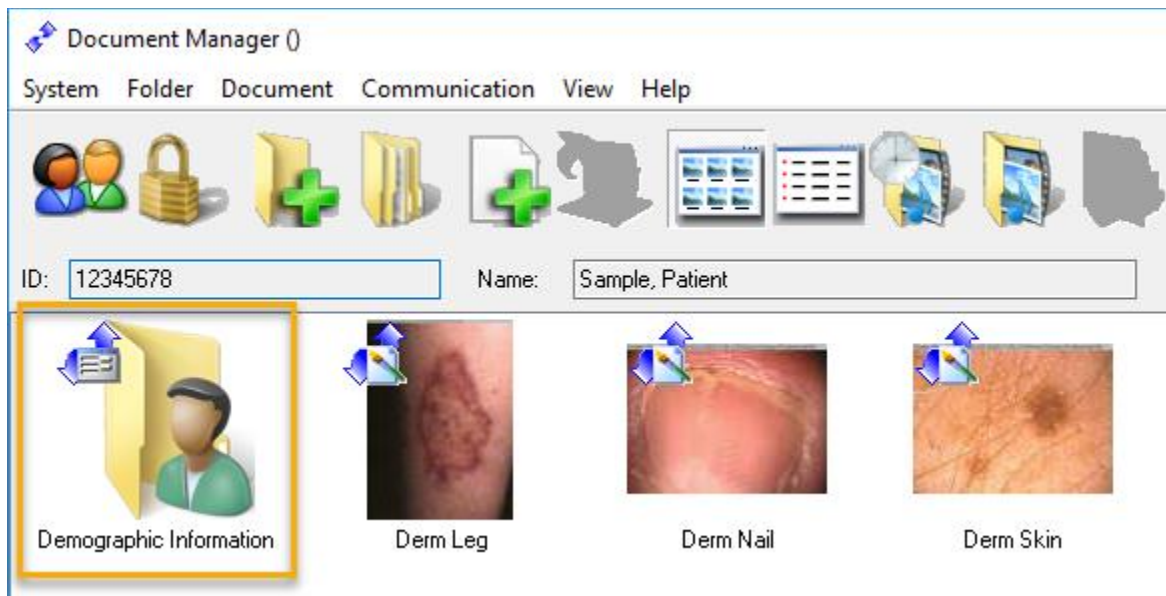

Currently the Folder database does not have a field defined for a **Secondary ID**. To overcome this limitation, we have created a workaround to allow the user to type in a **Secondary ID** and this **ID** is recognized in other programs such as our **Image Export Manager**.

If **Secondary IDs** are required for your use case, the default **Med01** demographics form needs to be replaced with a different one that contains the Secondary ID field. Below is an example form.

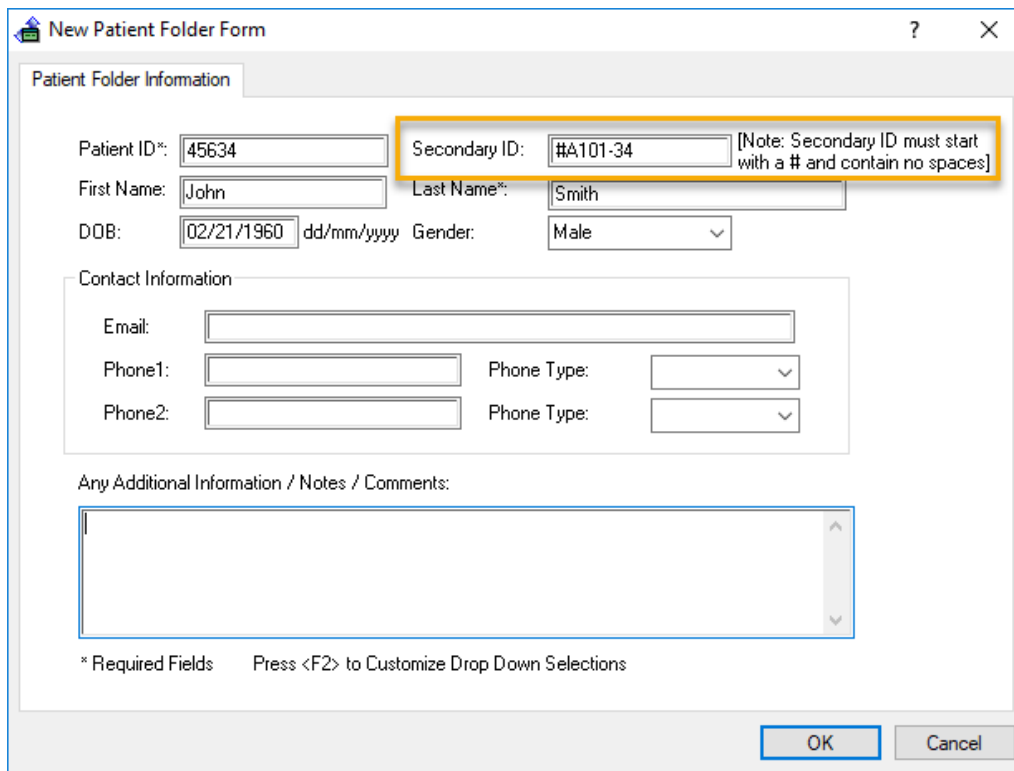

**New Patient Folder Form**

Patient Folder Information

Patient ID\*: 45634      Secondary ID: #A101-34 [Note: Secondary ID must start with a # and contain no spaces]

First Name: John      Last Name\*: Smith

DOB: 02/21/1960 dd/mm/yyyy      Gender: Male

Contact Information

Email:


Phone1:       Phone Type:

Phone2:       Phone Type:

Any Additional Information / Notes / Comments:

\* Required Fields      Press <F2> to Customize Drop Down Selections

OK      Cancel

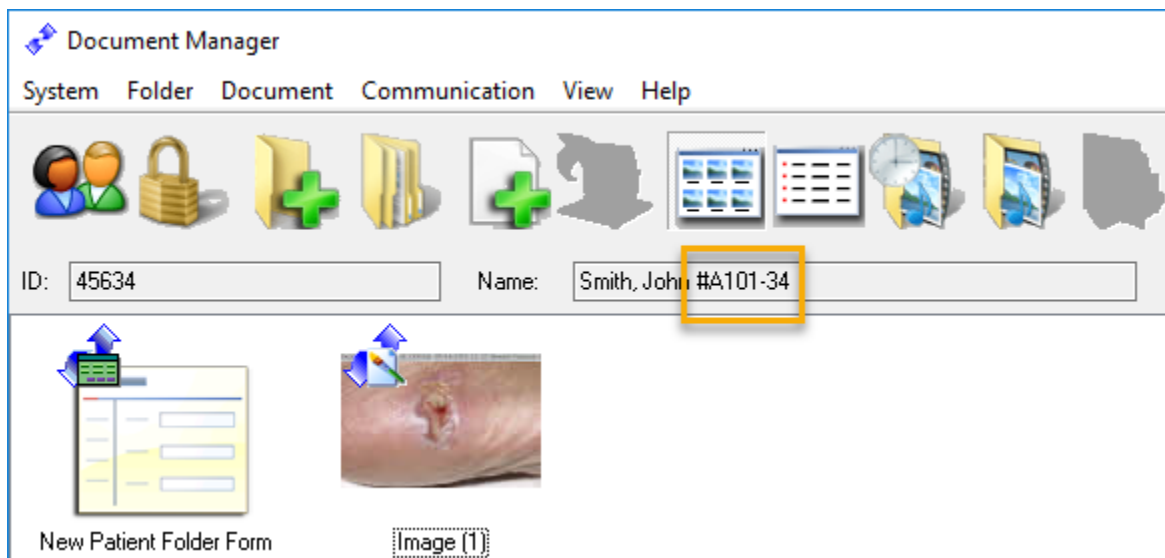


**Document Manager**

System   Folder   Document   Communication   View   Help

ID: 45634      Name: Smith, John #A101-34

New Patient Folder Form      Image (1)

To add a **Secondary ID** to an existing Folder, a special **Edit Folder** form needs to be installed. Contact Technical Support and they will help you get it installed.

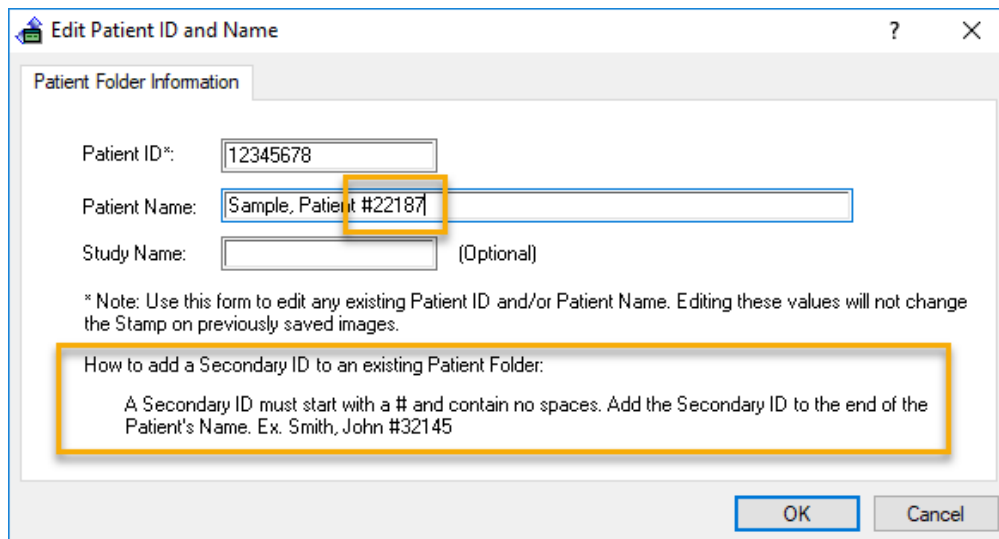

**Edit Patient ID and Name**

Patient Folder Information

Patient ID\*:

Patient Name:

Study Name:  (Optional)

\* Note: Use this form to edit any existing Patient ID and/or Patient Name. Editing these values will not change the Stamp on previously saved images.


How to add a Secondary ID to an existing Patient Folder:

A Secondary ID must start with a # and contain no spaces. Add the Secondary ID to the end of the Patient's Name. Ex. Smith, John #32145

OK Cancel

### Export Manager

The **Image Export Manager** is one of the programs that recognize the **Secondary ID** field. During the exporting of Images, the **Image Stamp** of an Image can be changed to include the **Secondary ID**. This is helpful when de-identifying an Image when being used in a Clinical Trial.

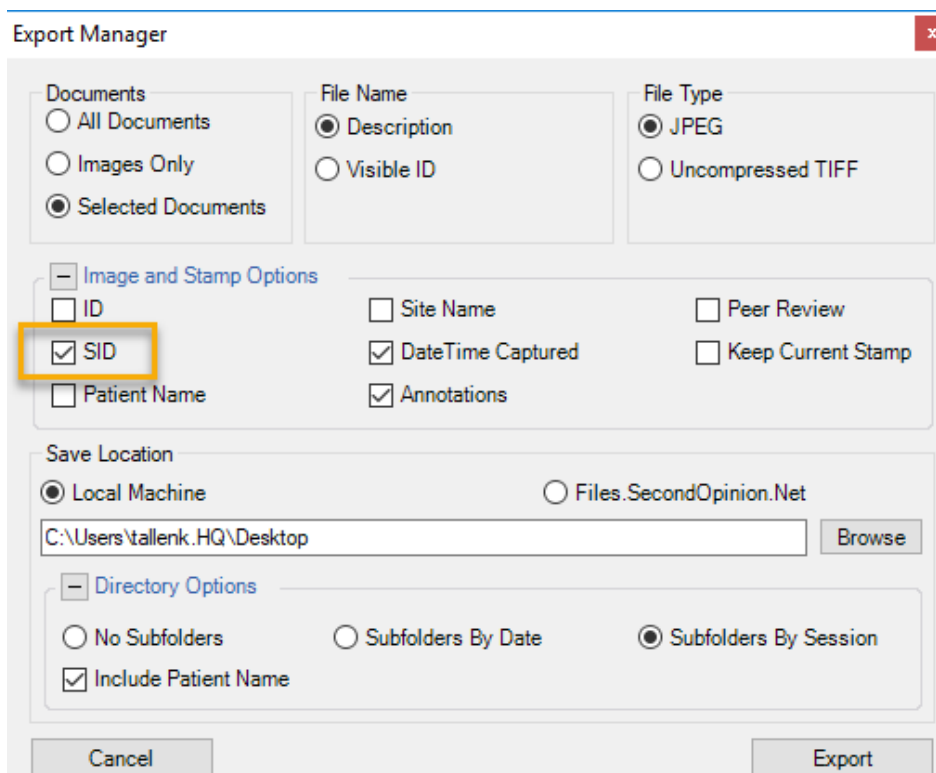

**Export Manager**

Documents

All Documents

Images Only

Selected Documents

File Name

Description

Visible ID

File Type

JPEG

Uncompressed TIFF

Image and Stamp Options

ID

SID

Patient Name

Site Name

Date/Time Captured

Annotations

Peer Review

Keep Current Stamp

Save Location

Local Machine

Files.SecondOpinion.Net

Browse

Directory Options

No Subfolders

Subfolders By Date

Subfolders By Session

Include Patient Name


Cancel Export

## Image Export Manager


The Image Export Manager is a simple to use tool to manage the process of exporting all or selected images from a Patient's folder to an unsecured folder outside of **Second Opinion™** Professional. During the export process the images are converted to an industry standard JPEG file format. These unsecured JPEG images can be used in a variety of different ways. They can be burned to a CD/DVD or copied to a memory card / flash drive for review by persons that do not have the **Second Opinion™** Image Viewer application. They can also be uploaded to an on-line image library / storage system.

**Note:** If you are only going to export selected images, select the images before running the **Image Export Manager**.

To start the application, **Click** on the **Export Manager** tool bar button or **Click** the **New Document** button, **Expand** the **Tools** section and **Click** on **Export Manager**.

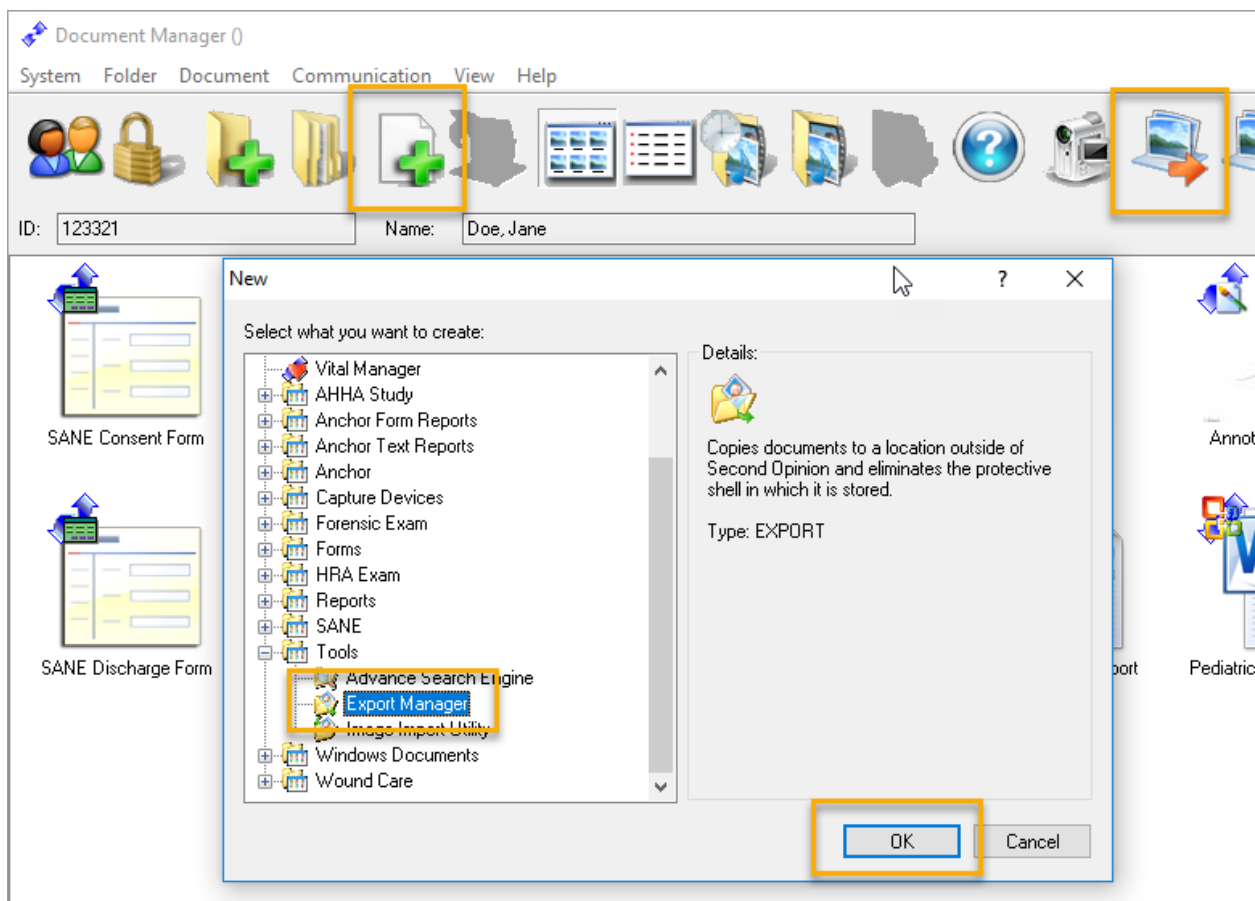


Before you can run the **Image Export Manager** you must acknowledge its disclaimer. Please review the disclaimer and if you do not understand any part of it, please call the Second Opinion Telemedicine office for clarification.

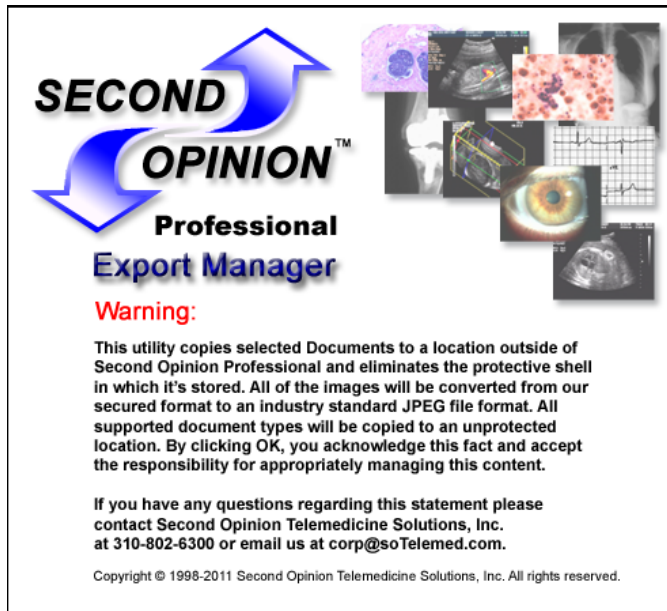

The Image and Stamp Options allow you change the image stamp on an imaged during the export process. As before, the Peer Review option removes / grays out the original image stamp.

The new Directory Options sections allows for the creation of sub folders based on the Date of the content or by Sessions that have been defined.

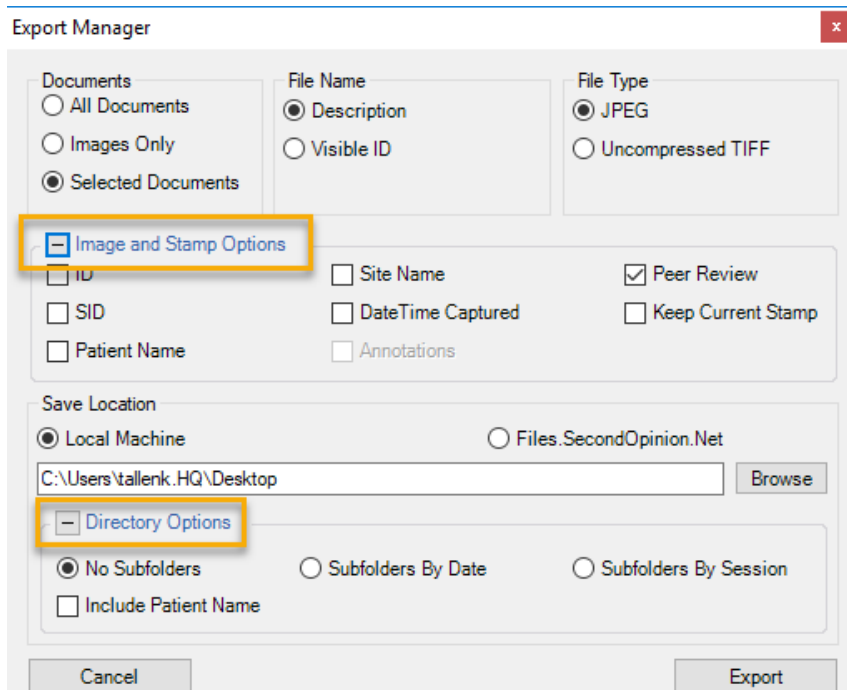



## Advanced Search Integration

This new Image Export Manager also works with the **Second Opinion™** Advanced Search Tool. The Advanced Search Tool now has an Export Manager Option. The query results (group of Patients) generated by the Advanced Search Tool can now be sent to the Image Export Manager and the result set can be exported in a tree like manner according to the defined choices.

Second Opinion Telemedicine Solutions, Inc.

20695 S. Western Ave. Suite 200

Torrance, CA 90501

310-802-6300

[www.sotelemed.com](http://www.sotelemed.com)

[corp@sotelemed.com](mailto:corp@sotelemed.com)

